

LiNA SeaStar™ and LiNA Skin Hook™

Complete single-use system for
retracting soft tissue from the
operative surface




Elastic Retraction System



Innovation in Gynecology

LiNA SeaStar™ and LiNA Skin Hook™

The LiNA SeaStar™ and LiNA Skin Hook™ offer a single-use solution for retracting delicate soft tissue. The flexible design of the system allows adjustment and repositioning of the LiNA SeaStar™ during the surgical procedures and provides exposure of the surgical site.

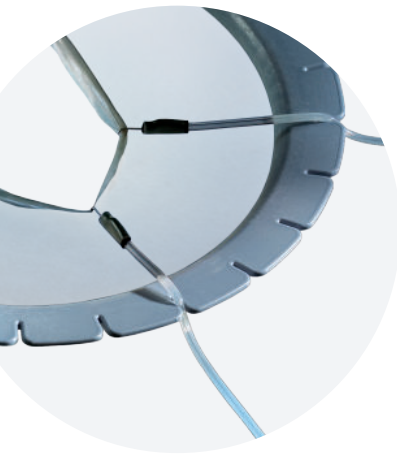

Visibility. Self-retaining.

Freeing the hands of the surgeon

The LiNA SeaStar™ and LiNA Skin Hook™ provide improved surgical space and visibility. Allowing you to focus on operating, teaching and training rather than retraction.


Versatility. Broad range of available options.

LiNA SeaStar™


The LiNA SeaStar™ is provided in a broad range of shapes and sizes. The frames can be adjusted to fit various anatomical areas, thereby facilitating different surgical applications e.g. vaginal and rectal surgery and other surgical disciplines.

LiNA Skin Hook™


The LiNA Skin Hook™ is available with four different stainless steel hook options: small sharp, small blunt, small angled extra sharp, large blunt. The flexible, elastic bands can be adjusted during the procedure to create optimal tension.

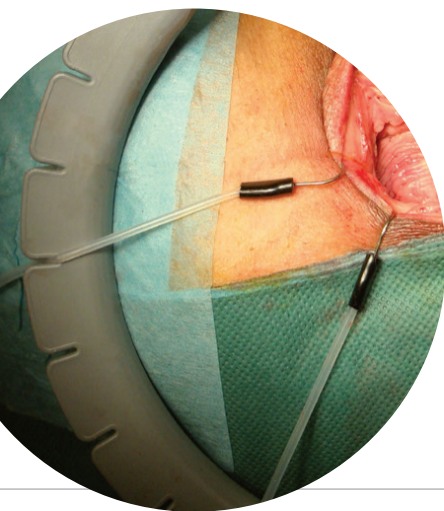


Prepare and drape the surgical area in usual manner. Apply the LiNA SeaStar™ according to the following steps:

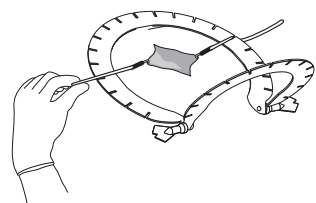

1. Place the LiNA SeaStar™ around the initial incision and insert the LiNA Skin Hook™ in directly opposing quadrants with equal tension.

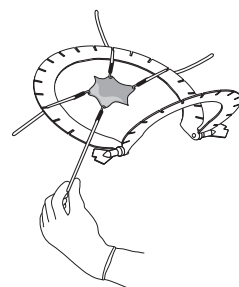

2. As dissection progresses, move the position of the LiNA Skin Hook™ into the deeper layers of fascia. You may then add new LiNA Skin Hooks or reposition those already in place to achieve full balanced retraction from any angle.

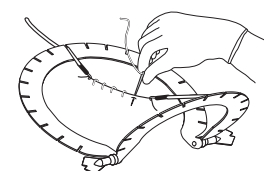

3. To ease closing, maintain constant opposing tension.



LiNA SeaStar™ and LiNA Skin Hook™


The LiNA SeaStar™ and LiNA Skin Hook™ are single-use.



REF 4000-S-SL



REF 4000-S-SS

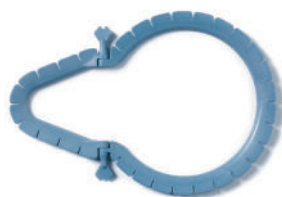

REF 4000-S-VL



REF 4000-S-VS



REF 4000-S-VV



REF 4000-S-Q

LiNA Skin Hooks
are actual size



REF SH-110
Small sharp hook



REF SH-120
Small blunt hook



REF SH-130
Small angled extra sharp hook



REF SH-220
Large blunt hook

Ordering information:

LiNA SeaStar™ and LiNA Skin Hook™



| Product Code | Description | UOM |
|--------------|--|--|
| 4000-S-SL | LiNA SeaStar™ – Single use retractor ring. SL model (Small/Large). | Box of 5, sterile |
| 4000-S-SS | LiNA SeaStar™ – Single use retractor ring. SS model (Small/Small). | Box of 5, sterile |
| 4000-S-VL | LiNA SeaStar™ – Single use retractor ring. VL model (V-shape/Large). | Box of 5, sterile |
| 4000-S-VS | LiNA SeaStar™ – Single use retractor ring. VS model (V-shape/Small). | Box of 5, sterile |
| 4000-S-VV | LiNA SeaStar™ – Single use retractor ring. VV model (V-shape/V-shape). | Box of 5, sterile |
| 4000-S-Q | LiNA SeaStar™ – Single use retractor ring. Squared model. | Box of 5, sterile |
| SH-110 | LiNA Skin Hook™ – Small sharp hook. Single use. | 2 pieces in a pouch, box of 5 pouches, sterile |
| SH-120 | LiNA Skin Hook™ – Small blunt hook. Single use. | 2 pieces in a pouch, box of 5 pouches, sterile |
| SH-130 | LiNA Skin Hook™ – Small angled extra sharp hook. Single use. | 2 pieces in a pouch, box of 5 pouches, sterile |
| SH-220 | LiNA Skin Hook™ – Large blunt hook. Single use. | 2 pieces in a pouch, box of 5 pouches, sterile |

Important: Please refer to the Instruction for Use for complete instructions, contraindications, warnings and precautions.

EU MDR 2017/745 devices class: SeaStar - I sterile, Skin Hook - IIa.
Name and number of the notified body: TUV Nord Polska sp. z o.o. # 2274.
©2025 LiNA Medical AG. All rights reserved.
LiNA is a registered trademark of LiNA Medical in EU, US and CN.

LiNA SeaStar and LiNA SkinHook / 2025-01 BV0056G en



LiNA Medical AG, Platz 3, 6039 Root D4 | Switzerland
Phone: +41 78 617 56 56, info@lina-medical.com | www.linamed.com



LiNA Medical Polska Sp. z o.o., Rolna 8A, Sady, 62-080 Tarnowo Podgórne | Poland
Phone: +48 61 222 21 21, info.poland@lina-medical.com | www.lina-medical.pl

EN ISO 13485:2016
Quality Management System

