LiNA Skin Hook™ (sterile)

Ref: SH-110, SH-120, SH-220, SH-130

en

Refer to the Symbol Definition Chart FV0223, the integral part of Instructions for Use, for the meanings of the symbols.

Product description:

The LiNA Skin Hook™ is together with LiNA SeaStar™ a complete system for retracting delicate soft tissue. By securing the LiNA Skin Hook™ on to the LiNA SeaStar™ the exposure of the incision is increased. The flexible and elastic hook can be adjusted during the procedure to create optimum tension.

The LiNA Skin Hook™ is available with 4 different stainless steel hooks.

Intended use:

The LiNA Skin Hook™ is intended to retract the soft tissue from the operative surface.

Indication for use:

The LiNA Skin Hook™ is intended to retract the soft tissue from the operative surface.

Patient population:

Patients intended to have surgical procedures which require optimal exposure.

Potential complications:

- Skin laceration.

Warnings:

- The LiNA Skin Hook™ is provided sterile using ethylene oxide sterilization. Carefully inspect the packaging for any damage prior to use. Do not attempt to use the device if product sterile barrier or its packaging is damaged.
- Do not use past expiration date.
- Do not use if the device is exposed to non-sterile surfaces before procedure.
- For single use only. Do not reuse, reprocess or re-sterilize LiNA Skin Hook™.
- Any reprocessing may impede the functions of this device. Reusing single use devices may also increase the risk of cross contamination. Attempts to clean the device results in risk of device malfunction.
- Carefully inspect the device before use. If any damage or defect is identified, dispose of the device.
- Do not use excessive force as it may cause product damage or user injury.
- No modification of this equipment is allowed.
- Special attention is needed when installing and removing LiNA Skin Hook™ from the operative surface since device has sharp edges.
- Failure to follow all instructions or any warnings or precautions may result in significant injury to the patient, physician or attendants and may have an adverse effect on the outcome of procedures performed. LiNA Skin Hook™ should not be used for any other purpose than intended function.

Precautions:

- Use of the LiNA Skin Hook™ requires adequate training and experience in performing surgical procedures. Intended user profile is a trained surgeon or physician.
- After use, the device is to be considered contaminated medical waste and may pose a potential infection risk. Dispose of the device and packaging in accordance with applicable laws, administrative, and/or local government policy and hospital procedures, including those regarding biohazards, microbiological hazards and infectious substances. Be aware of sharp edges.
- Please do not attach LiNA Skin Hook™ to home-made retractor rings.

Instruction for use:

- Remove LiNA Skin Hook™ from the pouch. Gently slide the silicone tube from the hook starting from the end of the tube. Do not use excessive force as it may cause product damage or user injury.
- Place the LiNA SeaStar™ around the initial incision and insert the LiNA Skin Hook™ in directly opposing quadrants with equal tension.
- As dissection progresses, move the position of the hook into the deeper layers of the fascia.
- You may then add new hooks or reposition those already in place to achieve full balanced retraction from any angle.
- To ease closing maintain constant opposing tension.

Reporting:

Any serious incident that has occurred in relation to the device should be reported to the LiNA Medical AG and the competent regulatory authority of the country in which the user and/or patient is established.

Caution:

Federal Law (USA) restricts this device to sale by or on the order of a trained physician.

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